# Bair Hugger® Therapy 700 Series Temperature Management Unit *Quick Reference*

**Frequency** = Every 12 months or 500 hours of use, or whenever service is performed.

# 1. Filter inspection

- a. Remove and inspect the filter.
- b. Return filter or replace as necessary.

Note: Replace every 500 hours or every 12 months (PN 90047).

#### 2. Calibrate the hose end sensor

- a. Attach the Bair Hugger test unit (PN 22110).
  - i. Connect temperature sensor to your meter.
- b. Run in ambient mode for 10 minutes.
- c. Enter the calibration mode.
  - i. Press the Alt (Bair Hugger logo) and Ambient button, together for approximately 3-seconds.
  - ii. Temperature measured at end of hose will display on LCD.
- d. Adjust Bair Hugger displayed temperature so it matches test unit sensor (± .1° increments).
  - i. Low button lowers displayed temperature.
  - ii. High button raises displayed temperature.

Note: Unit can only adjust  $\pm$  3.0°C. Try a different hose if more adjustment is required and call Arizant technical support with results.

e. Save Settings - Press the Alt (Bair Hugger logo) button to save your settings.

# 3. Test operating temperatures

- a. Run in 43°C (High), 38°C (Med) and 32°C (Low).
  - i. Measured temperatures should be within  $\pm$  1.5°C.

# 4. Test over temperature detection system

- a. Initiate O.T. test
  - i. Press the Alt (Bair Hugger logo) and High button, together for approximately 3-seconds.
  - ii. Wait as unit runs self test.
  - iii. Two beeps indicate test has completed.
- b. Review Results
  - i. Pass or fail indicated on display.

Complete Service Manuals are available at: www.bairhugger.com or http://www.arizant.com/arizant/manuals-faw.shtml

For Technical Service questions please call: 1-800-733-7775, select Option 2

©2009 Arizant Healthcare Inc. All rights reserved. 10393 West 70th Street, Eden Prairie, MN 55344 USA TEL 800-733-7775 • 952-947-1200 | FAX 800-776-0002 • 952-947-1400

Bair Hugger and the Bair Hugger logo are trademarks of Arizant Healthcare Inc., registered or pending in the U.S. Patent & Trademark office and in other countries. 602813B 12/09

