


Advanced skin care solutions to improve the lives of your patients

3M[™] Cavilon[™] Skin Care Solutions



Your best line of defense against conditions that can impede healing

You work hard to help your patients heal quickly and comfortably, but sometimes complications can slow their progress and negatively impact care costs and your facility's resources. Patient exposure to factors such as irritants, moisture, friction, shear and adhesives can lead to skin breakdown, including Moisture-Associated Skin Damage (MASD), Pressure Ulcer/Injury (PU/I) and Medical Adhesive-Related Skin Injury (MARSI).

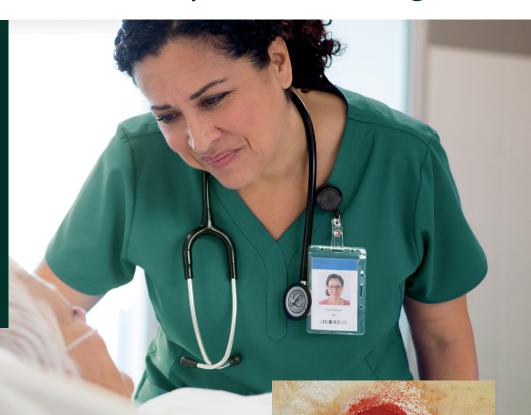

Moisture-Associated Skin Damage (MASD)

MASD is a term that describes several types of damage that occur when skin is exposed to excessive moisture and/or irritants. With over-hydration, the stratum corneum (outermost layer of the epidermis) becomes more permeable, skin pH becomes more alkaline, and inflammation occurs.


Although pressure, shear, and friction are widely recognized as threats to skin integrity, the significant risk posed by frequent exposure to excessive moisture often goes unnoticed in causing skin breakdown. It's crucial for clinicians to remain vigilant in preserving optimal skin conditions and in early identification and treatment of Moisture-Associated Skin Damage (MASD) to prevent its progression and subsequent skin breakdown.

Moisture and other fluids can lead to several types of MASD:

- Incontinence-Associated Dermatitis (IAD)
- Peristomal Skin Damage
- Periwound Skin Damage
- Intertriginous Dermatitis (ITD)





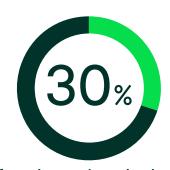

of oncology patients developed MARSI at their PICC insertion site over the course of two weeks⁴

Medical Adhesive-Related Skin Injury (MARSI)

Medical adhesives are a critical part of healthcare, but can cause Medical Adhesive-Related Skin Injury, or MARSI.

MARSI can cause pain, increase the risk of infection and delay healing — all of which can reduce a patient's/resident's quality of life.⁶ Skin tears, skin stripping and tension blisters are common but avoidable examples of MARSI.

One study showed a

21.4%

prevalence of MARSI in an acute care setting. Prevalence may vary by care setting and patient risk factors.






Зх

Patients with IAD are at an increased risk of superficial sacral pressure ulcers/injuries with an odds ratio of 2.99⁷

Pressure Ulcer/Injury (PU/I)

Skin breakdown can lead to a pressure ulcer/injury which is localized damage to the skin and underlying soft tissue. Moisture, friction and shear are accepted as risk factors for pressure ulcer/injury development.

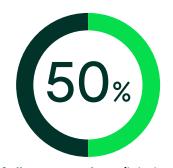

of all pressure ulcers/injuries occur at anatomical sites that overlay a bony prominence, such as the heel and sacrum^{8,9}

Simplifying your skin integrity needs with proven solutions

Set your level of care apart with solutions that set themselves apart from competitive products in test after test. With an advanced skin protectant, novel barriers and other products that help maintain skin health, we have the comprehensive solutions you need to protect the skin that protects your patients.



3M™ Cavilon™ Advanced Skin Protectant

Cavilon Advanced Skin Protectant featuring polymer-cyanoacrylate technology creates a highly durable, ultrathin barrier that attaches to wet, weepy¹⁰ or damaged skin and lasts up to seven days¹¹ — protecting patient skin.



3M™ Cavilon™ No Sting Barrier Film

With a unique terpolymer formula that delivers exceptional durability and flexibility, Cavilon No Sting Barrier Film is a gentle, effective and CHG-compatible solution for routine skin protection.








3M[™] Cavilon[™] Durable Barrier Cream

Cavilon Durable Barrier Cream is a durable, moisturizing barrier cream formulated with polymer technology that resists wash-off and allows adhesives to adhere.

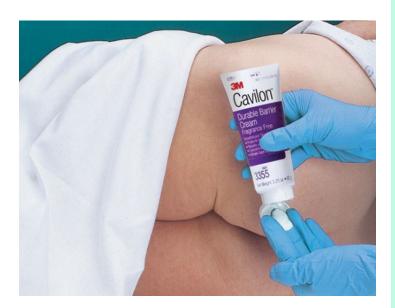



3M[™] Cavilon[™] No-Rinse Skin Cleanser


An important part of any skin integrity program, Cavilon No-Rinse Skin Cleanser is a gentle, effective cleanser that helps control odor. It's pH balanced and conditions and soothes skin.



3M[™] Cavilon[™] Extra Dry Skin Cream

Ideal for patients with diabetes or other dry skin conditions, the moisturizing effect of Cavilon Extra Dry Skin Cream lasts 24 hours to soften and relieve rough, cracked or scaly skin.

Find your optimal 3M™ Cavilon™ Skin Care solution

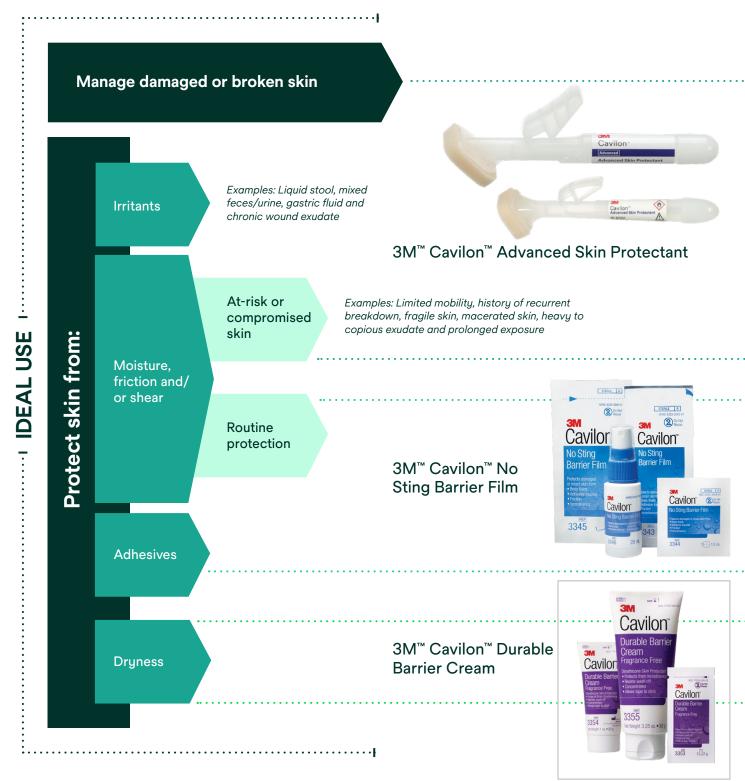

Finding the optimal solution for your skin integrity needs

	Moisture-Associated Skin Damage (MASD)					
	Incontinence- Associated Dermatitis (IAD)	Peristomal Skin Damage	Periwound Skin Damage	Intertriginous Dermatitis (ITD)	Pressure Ulcer/Injury	Medical Adhesive-Related Skin Injury (MARSI)
•••••	• • • • • • • • • • • • • • • • • • • •			• • • • • • • • • • • • • • • • • • • •	•••••	
Manage damaged or broken skin						
	Manage IAD	Manage peristomal/ perifistular skin damage	Manage periwound skin damage (e.g. maceration), including compatibility of use under 3M acrylic negative pressure drapes	Manage superficial skin damage from moisture and friction	Manage superficial skin injury in difficult-to-dress locations	Manage superficial skin damage (e.g. stripping, skin tears) from adhesive use, including compatibility of use under vascular access dressings
	Protect at-risk skin					
	Protect intact skin especially in the presence of diarrhea or mixed incontinence	Protect skin around problem fecal or urinary stomas, fistulas or tracheostomies	Protect skin around at-risk wounds (e.g. heavily draining wounds such as diabetic foot ulcers, venous leg ulcers or infected wounds)		Protect intact skin from moisture, friction or shear	
Ideal for routine protection						
		Protect intact skin for low-risk patients	Protect intact skin for low-risk patients	Protect intact skin from moisture and friction	Protect intact skin from moisture, friction or shear	Protect intact skin from adhesive products (e.g. tape, wound and vascular access dressings and ostomy products)


4

When you need to moisturize at-risk or severely dry skin, or protect skin at low risk from breaking down, consider using **3M**[™] **Cavilon Durable Barrier Cream**.

For more information, contact your Solventum Account Representative, call the Customer Helpline at 1-800-228-3957 or visit Solventum.com. These products can be ordered from your local distributor.

Ordering information



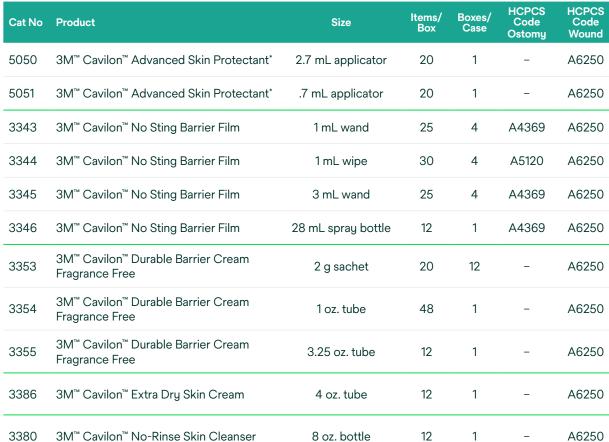

*RX only. See instructions for use.

HCPCS codes have been provided to assist you in the preparation of insurance claims. Please note, however, that the reimbursement information provided by Solventum and its representatives is intended to provide general information relevant to coverage and coding for 3M products. Insurers' reimbursement policies can vary and the use of the codes discussed here does not guarantee that an insurer will cover or pay at any particular level. Health care providers should exercise independent clinical judgement in choosing the codes which most accurately describe the products provided.

- 1. Gray M, Bartos S. Incontinence Associated Dermatitis in the Acute Care Setting: A Prospective Multi-site Epidemiologic Study. Presented at the 23rd Annual Meeting of the Wound Healing Society. 2013.
- 2. Nix D, Haugen V. Prevention and management of Incontinence-Associated Dermatitis. Drugs Aging. 2010. 27(6); 491-6.
- 3. Colwell JC, McNichol L, Boarini J. North America Wound, Ostomy, and Continence and Enterostomal Therapy Nurses Current Ostomy Care Practice Related to Peristomal Skin Issues. J Wound Ostomy Continence Nurs. 2017; 44(3): 257-261.
- 4. Zhao H, et al. Prevalence of medical adhesive-related skin injury at peripherally inserted central catheter insertion site in oncology patients. J Vasc Access. 2017 Nov 8:0. doi: 10.5301/jva.5000805.
- 5. Source: https://europepmc.org/abstract/med/15509110.
- 6. Cutting KF. Impact of adhesive surgical tape and wound dressing on the skin with reference to skin stripping. J Wound Care. 2008;157-158,160-162.
- 7. Demarre L, et al. Factors predicting the development of pressure ulcers in an at-risk population who receive standardized preventive care: secondary analyses of a multicentre randomised controlled trial. J Adv Nurs. 2015;71(2):391-403.
- 8. National Pressure Ulcer Advisory Panel, European Pressure Ulcer Advisory Panel and Pan Pacific Pressure Injury Alliance. Prevention and Treatment of Pressure Ulcers: Clinical Practice Guideline. Emily Haesler (Ed.). Cambridge Media: Osborne Park, Western Australia; 2014.
- 9. VanGuilder, C., MacFarlane, G. D., & Meyer, S. (2008). Results of Nine International Pressure Ulcer Prevalence Surveys: 1989 to 2005. Ostomy Wound Management, 54(2).
- 10. Brennan, Mary R.; Milne, Catherine T.; Agrell-Kann, Marie; Ekholm, Bruce P. Clinical Evaluation of a Skin Protectant for the Management of Incontinence-Associated Dermatitis: An Open-Label, Nonrandomized, Prospective Study. J of Wound, Ostomy & Continence Nurs. 2017; 44(2):172-180.
- 11. Solventum data on file. EM-05-305812.



Solventum Medical Surgical 3M Center, Building 275 2519 Conway Avenue East St. Paul, MN 55144-1000

Phone 1-800-275-4524 Web Solventum.com