



3M[™] Attest[™] Steam Chemical Integrator Guide

Recommended Placement

Wrapped Towel Pack or Gown, **Towel and Drape Pack**

Place CI in geometric center,



Manufacturer Supplied Container (multi-level, perforated tray, needs to be wrapped)*

Place CIs in center on each level.



Wrapped Instrument Tray

Place CI in geometric center, not on top.



Rigid Containers*

Place two CIs in opposite corners of each level.




Peel Pouches

When an internal CI is visible, an external Type 1 CI is not required.



IUSS Sterilization of Rigid Containers*

Place two CIs in opposite corners.

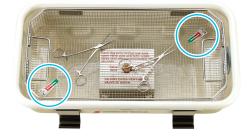


Interpretation

▲STEAM Type 5 Steam Chemical Integrator LOT 201910CX REF 1243

Processed — ACCEPT


Unprocessed

If the color bar reaches or crosses into the ACCEPT window, it is a pass — necessary conditions for sterilization have been met.

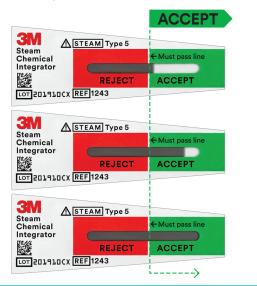

Processed — **REJECT**

If the color bar is in the REJECT region or on the line, it is considered a fail. The pack should be reprocessed and the cause

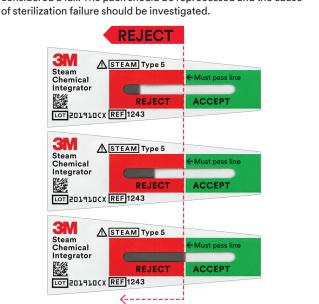

Contact your 3M Representative, call the 3M Health Care Help Line at 1-800-228-3957 or go.3M.com/steam

3M Health Care 3M Medical Solutions Division 3M Center, Building 275-4E-01 St. Paul, MN 55144 USA 1-800-228-3957 3M.com/medical

*The manufacturers of the device and container system should also be consulted for recommendation on number and placement of internal chemical indicators.

© 2020 3M. All rights reserved. 3M and Attest are marks and/or registered marks of 3M. Unauthorized use prohibited. Please recycle. Printed in USA. 70-2010-7298-3

ANSI/AAMI ST79:2017 Internal CIs should be placed

a) so that one CI is visible to the person opening the package;

b) in the area or areas considered least accessible to steam penetration; and

c) in accordance with all applicable written IFUs

3M[™] Attest[™] Steam Chemical Integrators, Type 5 conform with ISO 11140-1:2014.