

HOW TO USE

Curos Jet™

Disinfecting Cap for Needleless Connectors

Disinfects in 1 minute — protects ports for up to 7 days.





Push & Twist
the Curos Jet disinfecting cap
clockwise onto the end of the
needleless connector.

- If needleless connector is heavily soiled, clean with 70% isopropyl alcohol swab prior to using cap.
- Do not use cap if seal is broken, torn, or punctured.
- Cap must remain on needleless connector for minimum of one (1) minute and may remain on for seven (7) days if not removed.
- The needleless connector can be accessed immediately after removing the Curos Jet disinfecting cap that has been in place for a minimum of one (1) minute.
- Always place a new Curos Jet disinfecting cap on needleless connector following completion of a valve access.
- Dispose of the Curos Jet disinfecting cap after every use.
- Follow hospital protocol for disinfecting needleless connectors, if performing sequential accesses.

WARNING: To avoid potential for injury - use ONLY on needleless connectors.

CAUTION: Potential choking hazard.



Green means clean.



Critical & Chronic Care Solutions Division 3M Health Care 2510 Conway Avenue St. Paul, MN 55144 USA

St. Paul, MN 55144 USA

Please recycle this poster. Printed in U.S.A.

© 3M 2016. All rights reserved.

1800 228-3957

www.3M.com/C3SD

3M, Curos and Curos Jet are trademarks of 3M.

70-2011-5725-5



Active Ingredient: 70% isopropyl alcohol (IPA). Single Use Only. Rx Only. This device is not made with natural rubber latex or plasticiser Diethylhexylphthalate (DEHP). Refer to current device labeling for indications for use, warnings and product details.