




Product Guide

For Introducers

3M[™] Tegaderm[™] I.V. Advanced Securement Dressing 1685

Comprehensively designed to help you deliver exceptional patient care through advanced catheter securement, enhanced wear time and gentle removal.

Also available with antimicrobial protection

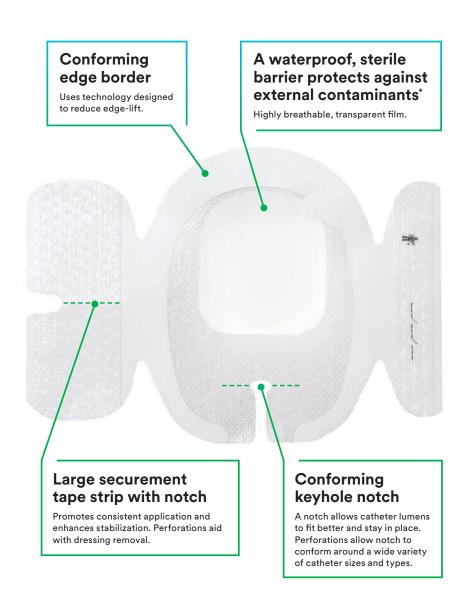

Site visibility

Transparent dressing enables early identification of complications at the insertion site.

Catheter securement


Designed to minimize catheter movement and dislodgement.


Ease of use

Frame delivery makes placement accurate and easy for consistent application.

Patient comfort

Safe to wear up to 7 days, potentially reducing the number of dressing changes and restarts.

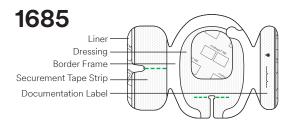


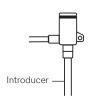

Product Guide

For Introducers

Prepare

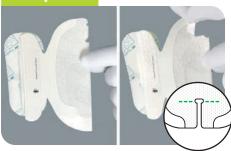

Prepare site according to facility protocol. Ensure site is completely dry. Open perforation(s) in keyhole notch before removing dressing liner.

Press



Do not stretch the dressing at placement. Peel the liner from the dressing and place the dressing window over insertion site, aligning extension tubing with securement notch. Apply firm pressure to entire dressing with one hand for optimal adhesion, while removing border frame with opposite hand.

Secure



Apply securement tape strip under the extension tubing and overlap back onto the dressing. Remove adhesive-free tabs.

Secure



Document the dressing change information on the label strip. Apply label strip on top of dressing, over catheter lumen(s). Remove adhesive-free tabs. Secure tubing with tape.

Monitor



Change the dressing:

- At least every 7 days
- If the dressing becomes loose, soiled or damp
- According to facility protocol



Open securement strip perforations.
Using a low and slow removal technique, start removing the dressing from where the catheter or tubing exits the dressing toward the catheter insertion site. Avoid skin trauma by peeling the dressing back rather than pulling it up from the skin.

Ordering Information

Product	Product Number	Overall Dressing Size	Dressings/Box	Boxes/Case
301	1685	3 ½ in x 4 ½ in	50	4

Refer to product Instructions for Use for other important information.



3M Health Care Medical Solutions Division 2510 Conway Avenue St. Paul, MN 55144-1000 USA



Visit 3M.com/IVtraining to find product training resources for all 3M IV care solutions.

Phone 1-800-228-3957 Web 3M.com/medical Please recycle. Printed in U.S.A. 3M and Tegaderm are marks and/or registered marks of 3M. Unauthorized use prohibited. © 3M 2018, 2020. All rights reserved. 70-2011-7177-7