


#### **Product Guide**

Compatible with BD Nexiva™ Closed IV Catheter System

### 3M™ Tegaderm™ I.V. Advanced Securement Dressing 1882

Comprehensively designed to help you deliver exceptional patient care through advanced catheter securement, enhanced wear time and gentle removal. This dressing is an option for smaller vascular access sites in pediatrics and adults and is safe to wear up to 7 days.

Also available with antimicrobial protection

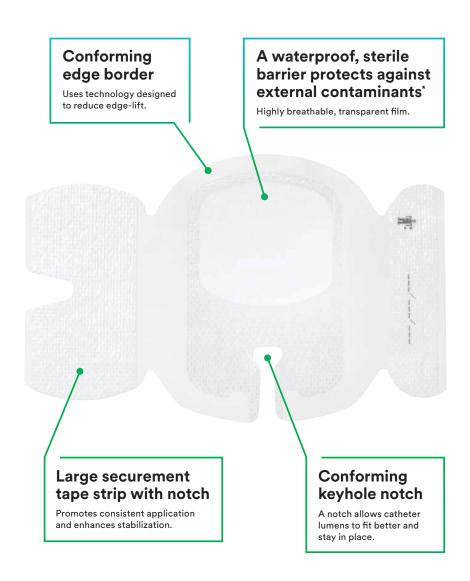

## Site visibility

Transparent dressing enables early identification of complications at the insertion site.

## Catheter securement


Designed to minimize catheter movement and dislodgement.

## Ease of use

Frame delivery makes placement accurate and easy for consistent application.

## Patient comfort

Safe to wear up to 7 days, potentially reducing the number of dressing changes and restarts.

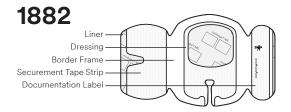

#### **Product Guide**

Compatible with BD Nexiva™ Closed IV Catheter System





Prepare site according to facility protocol. Ensure site is completely dry.

# Press One-handed application technique

Do not stretch the dressing at placement. Peel the liner from the dressing and place the dressing window over insertion site, aligning extension tubing with securement notch. Apply firm pressure to entire dressing with one hand for optimal adhesion, while removing border frame with opposite hand.

#### Secure



Apply securement tape strip under the extension tubing and overlap back onto the dressing. Remove adhesive-free tabs.

#### Secure



Document the dressing change information on the label strip. Apply label strip on top of dressing, over catheter lumen(s). Remove adhesive-free tabs. Secure tubing with tape.

#### **Monitor**



Change the dressing:

- At least every 7 days
- If the dressing becomes loose, soiled or damp
- · According to facility protocol



Remove tape strips. Using a low and slow removal technique, start removing the dressing from where the catheter or tubing exits the dressing toward the catheter insertion site. Avoid skin trauma by peeling the dressing back rather than pulling it up from the skin.

#### Ordering Information

Refer to product Instructions for Use for other important information.



3M Health Care Medical Solutions Division 2510 Conway Avenue St. Paul, MN 55144-1000 USA Product Product Number Overall Dressing Size Dressings/Box Boxes/Case

1882 2 ¾ in x 3 ¾ in 50 4



Visit 3M.com/IVtraining to find product training resources for all 3M IV care solutions.

Phone 1-800-228-3957 Web 3M.com/medical Please recycle. Printed in U.S.A.
3M and Tegaderm are marks and/or registered marks of 3M. Unauthorized use prohibited.
© 3M 2020. All rights reserved.
BD Nexiva is a trademark of BD.
70-2011-7966-3