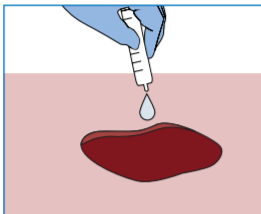
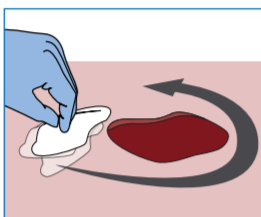



3M™ Snap™ Therapy System Application Guide

How to Apply

- 1** Prepare the wound bed and periwound skin per institutional protocol and irrigate wound bed thoroughly with normal saline.


- 2** If necessary, apply a skin protectant to the surrounding skin and allow to dry.


- 3** If necessary, cut a single layer of wide-mesh, non-adherent dressing to the size of the wound and place on the wound bed.

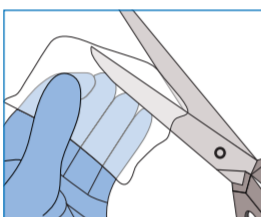

- 4** Cut the foam interface to fit the size and shape of the wound.


- 5** Place foam interface into wound cavity. Foam interface should fill the wound cavity and extend above the wound margins.



- 6** If necessary, remove the 3M™ Snap™ SecurRing™ Hydrocolloid Skin Barrier from packaging. Remove release liner from both surfaces.


- 7** If necessary, gently mold, stretch, compress or roll the 3M™ Snap™ SecurRing™ Hydrocolloid Skin Barrier to any shape to fit around the outside of the wound, leaving approx 1cm of intact skin between the 3M™ Snap™ SecurRing™ hydrocolloid skin barrier and the wound margin.


- 8** Remove backing and place the 3M™ Snap™ Advanced Dressing over the wound and seal.


- 9** At a right angle, cut the dressing tubing to the desired length, if required.


- 10** Fully insert the tube fitting into the tubing.

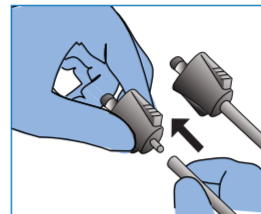
- 11** Connect the 3M™ Snap™ Therapy Cartridge to the tube fitting using both hands.


- 12** To activate the 3M™ Snap™ Therapy Cartridge, push down the activation/reset key to release and remove the key from the Snap therapy cartridge. Cartridge may need to be primed multiple times depending on size of the dressing.



- 13** Check negative pressure operation. The 3M™ Snap™ Therapy System is working properly if:

 - Green capacity indicator is both visible and stationary in the chamber window
 - Dressing has a "sucked down" appearance
 - Dressing feels hard to the touch
- 14** Secure the 3M™ Snap™ Therapy Cartridge to the patient's extremity or belt using the 3M™ Snap™ Therapy Strap (if being utilized).




Tips & Tricks For Challenging Anatomical Areas


Heel Application

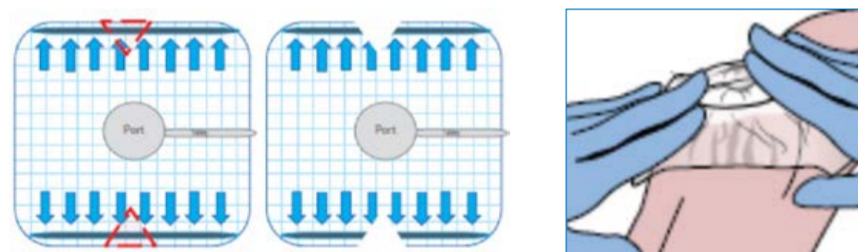


Toe Application I

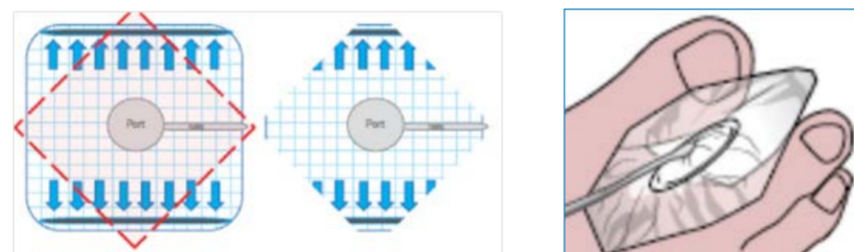



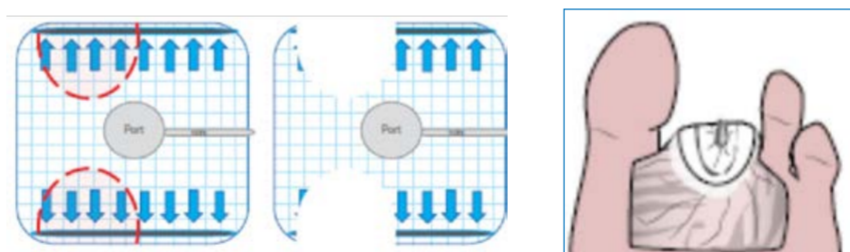



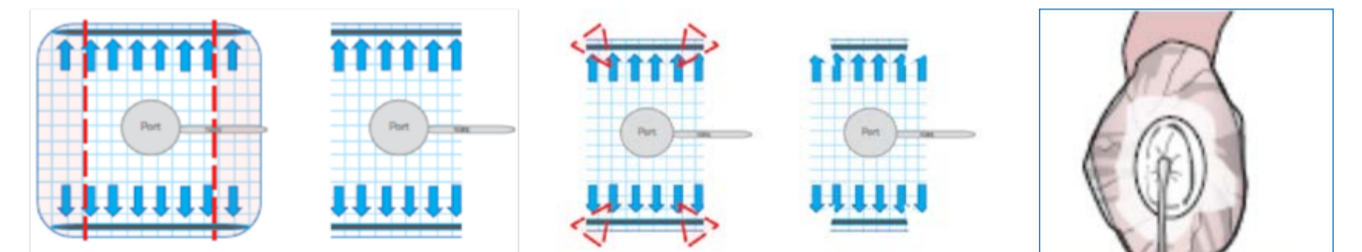



Toe Application II



Transmetatarsal Amputation (TMA):



Ordering Information

3M™ Snap™ Therapy Cartridge				3M™ Snap™ Therapy Strap	3M™ Snap™ Plus Therapy Strap	3M™ Snap™ Advanced Dressing Kit		3M™ Snap™ Bridge Dressing Kit	3M™ Snap™ SecurRing™ Hydrocolloid Skin Barrier
									
SKU: SNPA125P Pressure: -125mmHg Capacity: 150ml	SKU: SNPA125 Pressure: -125mmHg Capacity: 60ml	SKU: SNPA100 Pressure: -100mmHg Capacity: 60ml	SKU: SNPA075 Pressure: -75mmHg Capacity: 60ml	SKU: STPAS Size: Small 46cm SKU: STPAM Size: Medium 53cm SKU: STPAL Size: Large 61cm	SKU: STPASP Size: Small 46cm SKU: STPAMP Size: Medium 53cm SKU: STPALP Size: Large 61cm	SKU: SKTF10X10 Size: 10cm x 10cm Interface: Reticulated Open Cell Foam (blue)	SKU: SKTF15X15 Size: 15cm x 15cm Interface: Reticulated Open Cell Foam (blue)	SKU: BKTF14X11 Size: 14cm x 11cm Interface: Foam SKU: BKTF14X11S Size: 14cm x 11cm with 3M™ Snap™ SecurRing™ Hydrocolloid Skin Barrier Interface: Foam	SKU: SRNG10 Size: 5cm diameter

For additional information, please contact your local representative.

NOTE: Specific indications, contraindications, warnings, precautions and safety information exist for these products and therapies. Please consult a clinician and product instructions for use prior to application. This material is intended for healthcare professionals.

© 2021 3M. All rights reserved. 3M and the other marks shown are marks and/or registered marks. Unauthorized use prohibited. Used under license in Canada. 70-2011-8244-4 PRA-PM-EU-00416 (05/21)