



Cavilon™

Skin Care Solutions

3M™ Cavilon™ Advanced Skin Protectant

Protect at-risk or damaged skin when using critical vascular access dressings.

Delivering reliable under-dressing protection

Without the proper protection at the vascular site, factors beyond adhesive-related skin injuries (MARS), like weeping or erythema, put patients at risk of catheter-associated skin damage (CASD).



Cavilon Advanced Skin Protectant:



Adheres to wet, weepy, damaged skin, creating an environment that allows healing.¹



Does not diminish adherence of adhesive products.



Is breathable, so moisture can evaporate.



Compatible with Chlorhexidine Gluconate (CHG) skin prep.*



Take a proactive approach to protecting skin around a vascular access device.

Precautions: The product can increase the adhesion of some adhesive products (e.g., vascular access dressings), particularly in the first few days of use. When removing an adhesive product, it is important to exercise care. Use an adhesive remover containing HMDS to remove an adhesive product (including 3M acrylic negative pressure drapes) that has been placed over the film. This is especially important for patients with fragile skin.

*If using a securement dressing containing Chlorhexidine Gluconate (CHG), do not apply film to skin intended to come in contact with the CHG gel pad.

Unique Solutions for Skin Protection.

3M™ Cavilon™ Advanced Skin Protectant



3M™ Cavilon™ No Sting Barrier Film



Use for routine protection around IV sites.	<input type="checkbox"/>	<input checked="" type="checkbox"/>
Does not minimize adhesion.	<input checked="" type="checkbox"/>	<input checked="" type="checkbox"/>
Product is housed in sterile packaging.	<input checked="" type="checkbox"/>	<input checked="" type="checkbox"/> *
Recommended for use on at-risk and broken, wet/weepy, denuded skin.	<input checked="" type="checkbox"/>	<input type="checkbox"/>
Use for routine skin protection from adhesive products.	<input type="checkbox"/>	<input checked="" type="checkbox"/>

Note: Sterility is on par with other cyanoacrylate-based skin preps.

*Applies to wipes and swabs only

3M™ Cavilon™ Advanced Skin Protectant is available in two sizes to suit a variety of applications.



3M™ Cavilon™ Advanced Skin Protectant 5050



3M™ Cavilon™ Advanced Skin Protectant 5051



Ordering Information

Cat. No	Size	Approx. Coverage	Ideal for	Items/Box
5050	2.7ml applicator	100 sq. in.	<ul style="list-style-type: none"> Incontinence-associated skin damage protection and management 	20
5051	0.7ml applicator	25 sq. in.	<ul style="list-style-type: none"> Peristomal, peri-tube, peri-fistula, and periwound skin damage protection and management Protect skin around IV insertion site 	20

For more information on Cavilon Advanced Skin Protectant, go to [3M.com/Cavilon](https://www.3M.com/Cavilon) or contact your 3M account representative.

Note: Specific indications, limitations, contraindications, warnings, precautions and safety information exist for these products and therapies some of which may be Rx only. Please consult a clinician and product instructions for use prior to application.



3M Company
2510 Conway Ave
St. Paul, MN 55144 USA

Phone 1-800-275-4524 (NPWT products)
1-800-228-3957
Web 3M.com/medical

1. Acton C, Ivins N, Bainbridge P, Browning P. Management of incontinence-associated dermatitis patients using a skin protectant in acute care: a case series. J Wound Care. 2020;29(1):18-26. doi:10.12968/jowc.2020.29.1.18

© 2023 3M. All rights reserved. 3M and the other marks shown are marks and/or registered marks. Unauthorized use prohibited. Under license in Canada. GL_70-2013-1570-5