

IDEAL USE |

Protect skin from:

Manage damaged or broken skin

Irritants

Examples:
Liquid stool,
mixed feces/urine,
gastric fluid and
chronic wound
exudate

Moisture,
friction and/
or shear

At-risk or
compromised
skin

Routine
protection

Adhesives

Dryness

3M™ Cavilon™
Advanced Skin Protectant



Examples: Limited mobility, history of recurrent breakdown, fragile skin,
macerated skin, heavy to copious exudate and prolonged exposure

3M™ Cavilon™ No
Sting Barrier Film



3M™ Cavilon™ Durable Barrier Cream



Finding the optimal solution for your skin integrity needs


Moisture-Associated Skin Damage (MASD)



Incontinence-Associated Dermatitis (IAD)



Peristomal Skin Damage

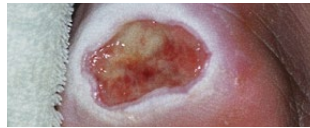


Periwound Skin Damage




Intertriginous Dermatitis (ITD)

Pressure Ulcer/Injury

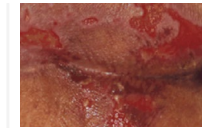

Medical Adhesive-Related Skin Injury (MARS)

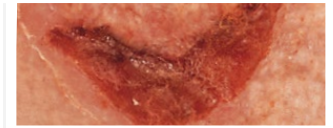

3M™ Cavilon™ Advanced Skin Protectant



Manage damaged or broken skin

Manage IAD

Manage peristomal/perifistular skin damage

Manage periwound skin damage (e.g. maceration), including compatibility of use under 3M acrylic negative pressure drapes

Manage superficial skin damage from moisture and friction

Manage superficial skin injury in difficult-to-dress locations

Manage superficial skin damage (e.g. stripping, skin tears) from adhesive use, including compatibility of use under vascular access dressings

Protect at-risk skin

Protect intact skin especially in the presence of diarrhea or mixed incontinence

Protect skin around problem fecal or urinary stomas, fistulas or tracheostomies

Protect skin around at-risk wounds (e.g. heavily draining wounds such as diabetic foot ulcers, venous leg ulcers or infected wounds)

Protect intact skin from moisture, friction or shear

Ideal for routine protection

Protect intact skin for low-risk patients

Protect intact skin for low-risk patients

Protect intact skin from moisture and friction

Protect intact skin from moisture, friction or shear

Protect intact skin from adhesive products (e.g. tape, wound and vascular access dressings and ostomy products)



3M™ Cavilon™ No Sting Barrier Film



Solventum Medical Surgical

3M Center, Building 275
2519 Conway Avenue East
St. Paul, MN 55144-1000

Phone 1-800-275-4524
Web Solventum.com

© Solventum 2024. All Rights Reserved. Solventum and the S logo are trademarks of Solventum or its affiliates. 3M and the 3M logo are trademarks of 3M. Other trademarks are the property of their respective owners. US_70-2011-7075-3



When you need to moisturize at-risk or severely dry skin, or protect skin at low risk from breaking down, consider using **3M™ Cavilon™ Durable Barrier Cream.**