



Prevena[™]

Single Use Negative Pressure Wound Therapy

PRM


Proactive Risk Management (PRM)
with 3M™ Prevena™ Therapy

Vascular Surgery

Advancing the standard of care

Helping to protect vascular
surgery incisions beyond the OR



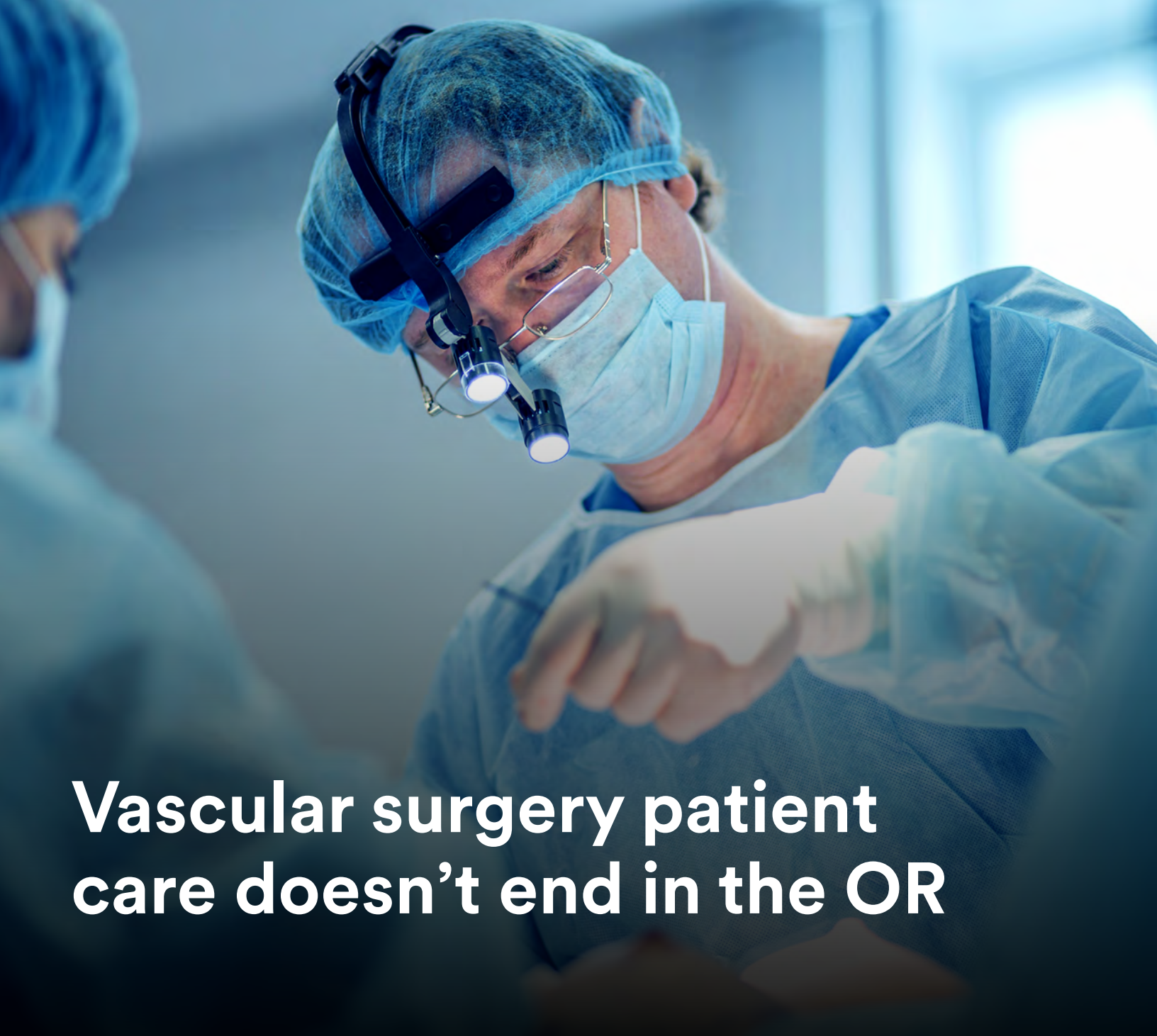

Vascular surgery patient care doesn't end in the OR

In an increasingly overwhelmed healthcare system, healthcare providers are asked to do more with fewer resources. Combined with the increasing prevalence of comorbidities and other risk factors adds to the potential for postoperative complications beyond the operating room despite best-in-class care provided.

Postoperative complications can create a ripple effect of consequences, like disrupted healing, extended hospital stays, and poor patient outcomes. Today's complex care environment makes protecting against the ripple effect of these complications a high priority.

Recognizing the risks of vascular surgery

Your confidence in the OR can help give patients peace of mind. However, that confidence can be lost when your patient is discharged. At that point there can be multiple concerns, including swelling, infections and improper tissue integration. 3M can help deliver positive outcomes for you and your patient.

The impact of complications from groin incisions/vascular bypass

Groin incisions are commonly used in vascular surgery to expose the femoral vessels and are used for many frequently performed procedures.

Given the groin's proximity to the external genitalia and skin flora, groin surgical site infections can commonly occur during vascular surgery. Risk increases if compounded by other patient-related factors such as diabetes, smoking, obesity, etc.



SSI rate following vascular groin incisions¹⁻⁵

\$11,973

Mean incremental cost to treat surgical site complications following open lower extremity vascular surgery⁶


The cost of surgical site infections (SSIs):



Patients affected each year⁷



Readmission is necessary 5x more often⁸

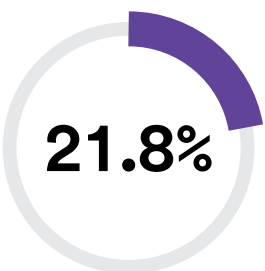

SSIs percentage of all healthcare-associated infections (HAIs)⁹



Average increase in length of hospital stay from SSIs¹⁰

+\$38,656

Average added cost from SSIs¹⁰

\$3.3 Billion

Cost to the healthcare system per year⁷

Extend your control over postoperative healing.

3M™ Prevena™ Therapy is the first closed-incision negative pressure therapy (ciNPT) solution of its kind to help reduce the risk or incidence of seromas and superficial surgical site infections (SSIs) in Class I and II wounds.* It helps protect the incision site after surgery up to 7 days — extending your control over postoperative healing and helping patients at risk of developing complications.

Prevena Therapy offers surgeons the confidence to help protect patients beyond the OR.



Acting as a barrier to external contamination



Delivering continuous -125 mmHg up to 7 days



Helping to hold incision edges together



Decreasing lateral tension of sutured/stapled incisions¹¹



Removing fluids and infectious materials^{**}



Reducing edema




*The effectiveness of Prevena Therapy in reducing the incidence of SSIs and seroma in all surgical procedures and populations has not been demonstrated. See full indications for use and limitations at HCBGRegulatory.3M.com.

**In a canister.

The advanced science of 3M™ Prevena™ Therapy

Prevena Therapy utilizes continuous -125 mmHg negative pressure therapy, reticulated open cell foam (ROCF) dressing technology, and optimized exudate management (replaceable canister) over closed incisions to help enhance healing.

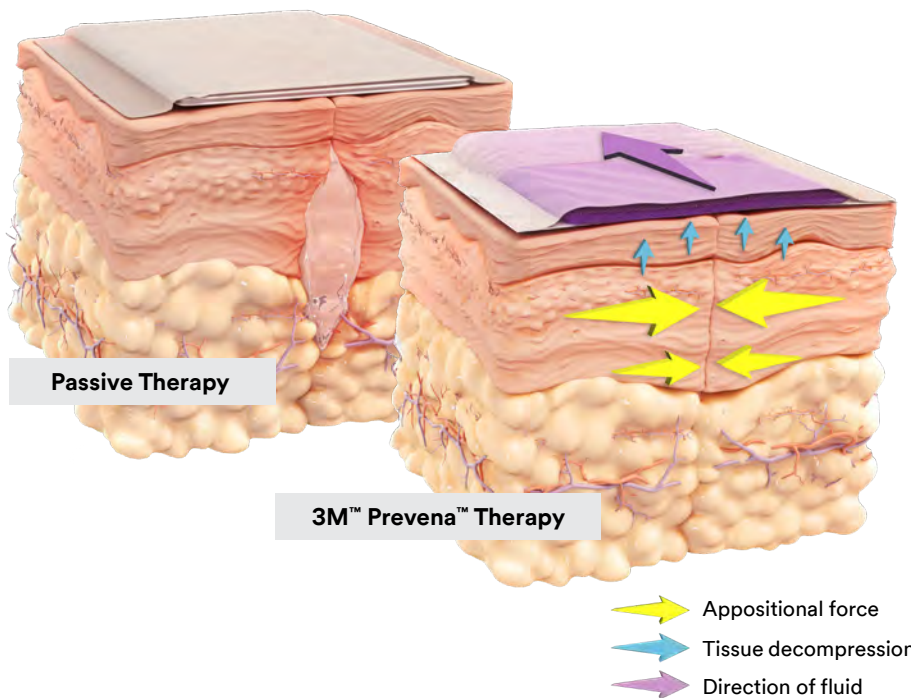

Designed to help optimize postoperative care:


- Contours allow for even distribution of negative pressure
- Bolster and stabilize the incision and surrounding soft tissue
- Enhanced fluid management*
- Conforms to allow movement

How Prevena Therapy may help reduce postoperative edema**

The effects of negative pressure therapy applied to intact skin were evaluated using finite element analysis (FEA).¹²

Based on the analysis, it is hypothesized that negative pressure therapy on intact skin provides volumetric expansion that may help:

- Expand the tissue beneath the dressing, pulling the tissue open
- Lower local interstitial fluid pressure
- Open lymphatics to allow fluid clearance

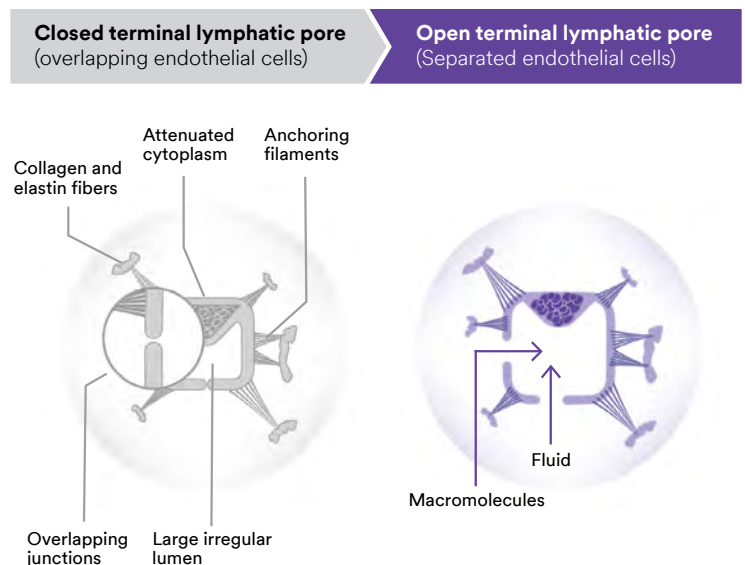

*In a canister.

**Information contained within conducted bench and animal studies have not been evaluated by the U.S. Food & Drug Administration. Results have not been confirmed in human studies.

Patients and procedures that may benefit from 3M™ Prevena™ Therapy

A multidisciplinary group of surgical and infectious disease experts developed an algorithm to guide when to consider using closed-incision negative pressure therapy (ciNPT).¹³ They recommend that surgeons consider using ciNPT for patients at high risk for developing surgical site occurrences (SSOs) or who are undergoing a high-risk procedure or a procedure that would have highly morbid consequences if a surgical site infection (SSI) occurred.


Consensus recommendations based on:

- Literature review
- ciNPT experiences
- Known risk factors for SSOs

Findings:

- Numerous publications reported SSI risk factors, with the most common including obesity (body mass index ≥ 30 kg/m²); diabetes mellitus; tobacco use; or prolonged surgical time
- It is recommended that the surgeon assess the individual patient's risk factors and surgical risks

Risk factor assessment for ciNPT

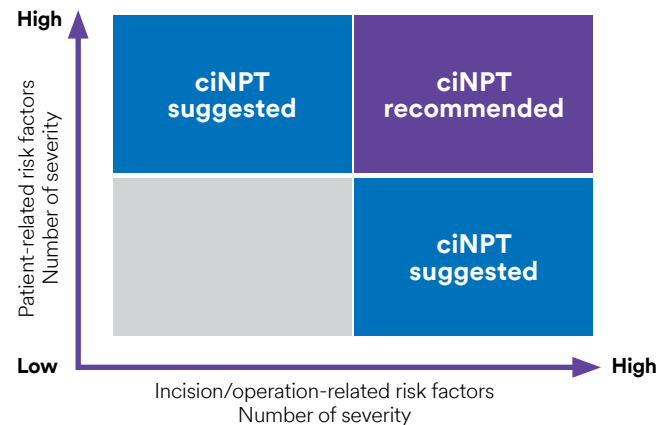

Additional factors to consider:

Patient-related risk factors		Patient-related risk factors	
<ul style="list-style-type: none"> • Diabetes mellitus • Acetylsalicylic acid Score ≥ 3 • Advanced age • Obesity • Active tobacco use • Hypoalbuminemia • Corticosteroid usage 	<ul style="list-style-type: none"> • Active alcoholism • Male sex • Hematoma • Chronic renal insufficiency • Chronic obstructive pulmonary disease 	<ul style="list-style-type: none"> • High tension incision • Repeated incisions • Extensive undermining • Traumatized soft tissue • Edema • Contamination • Emergency procedure 	<ul style="list-style-type: none"> • Prolonged operation time • Post-surgical radiation • Mechanically unfavorable site

Procedure/operation-related risk factors:

General	Plastic	Orthopedic	Vascular	Cardiovascular
<ul style="list-style-type: none"> • Open general • Open colorectal • Open urology • Open obstetrics/gynecology • Incisional hernia repair 	<ul style="list-style-type: none"> • Post-bariatric abdominoplasty • Breast reconstruction • Big soft tissue defects • Soilage risk 	<ul style="list-style-type: none"> • Open reduction and internal fixation of fractures • Fasciotomy • Above/below knee amputation 	<ul style="list-style-type: none"> • Above/below knee amputation • Synthetic graft implantations 	<ul style="list-style-type: none"> • Sternotomy

FDA indications for use

3M™ Prevena™ 125 Therapy Unit and 3M™ Prevena™ Plus 125 Therapy Unit manage the environment of closed surgical incisions and remove fluid away from the surgical incision via the application of -125 mmHg continuous negative pressure. When used with legally marketed compatible dressings, Prevena 125 Therapy Unit and Prevena Plus 125 Therapy Unit are intended to aid in reducing the incidence of seroma and, in patients at high risk for post-operative infections, aid in reducing the incidence of superficial surgical site infection in Class I and Class II wounds.

Note: The effectiveness of Prevena Therapy in reducing the incidence of SSIs and seroma in all surgical procedures and populations has not been demonstrated. See full indications for use and limitations at HCBGRegulatory.3M.com.




Clinical evidence supporting the use of 3M™ Prevena™ Therapy is growing

Peer-reviewed meta-analysis of 6 studies for vascular surgery procedures demonstrated **3M™ Prevena™ Therapy helped significantly reduce the risk of surgical site infections (SSIs), revision surgeries, and reduced hospital length of stay for groin incisions.**¹⁴


Reduced clinical complication risk:



Surgical site infections (SSIs)
6 studies; $p < 0.001^*$

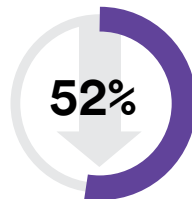

SSIs - Szilagyi I
4 studies; $p = 0.001^*$

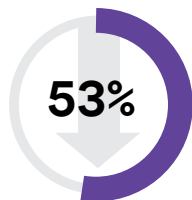

Revision surgeries
4 studies; $p = 0.02^*$



Hospital length of stay
2 studies; $p = 0.01^*$

For more clinical evidence, scan the code below to visit our clinical evidence compendium.



*Statistically significant ($p < 0.05$).
Calculation(s) are derived based on the relative patient group incidence rate reported in this study.

Rethink incision management

3M™ Prevena™ and Prevena Restor™ Dressings help protect the incision while providing even distribution of negative pressure therapy to help reduce the risk of complications, optimize the healing environment, and enhance the recovery experience.

Prevena Dressings and Prevena Restor Dressings can be applied to various procedures and anatomical locations.

3M™ Prevena™ Dressings

Linear Coverage

Prevena Dressings help protect and manage the environment of closed surgical incisions and remove fluid away from the surgical incision via the application of continuous negative pressure.



Linear coverage

Designed to manage linear, intersecting, and multiple incisions

Pre-shaped designs for peel and place application



3M™ Prevena™ Peel and Place Dressing – 13 cm



3M™ Prevena™ Peel and Place Dressing – 20 cm



3M™ Prevena™ Plus Peel and Place Dressing – 35 cm

Designed for flexibility and user customization



3M™ Prevena™ Customizable Dressing



Designed to perform for you and your patients

Acts as a barrier

Designed to minimize external contamination to help reduce surgical site complications

Precision designed

Dressings seamlessly conform to the patient for even distribution of negative pressure and allows for patient movement

Easy to use

A variety of peel and place dressings are available, plus customizable options


Shower-friendly

See Prevena Therapy Patient and Clinician Guides for additional details

3M™ Prevena Restor™ Dressings

Area Coverage

Prevena Restor Dressings offer comprehensive protection for a variety of anatomical locations. When combined with negative pressure, the dressings help provide stabilization of the incision and protect the surgical site and surrounding soft tissue.

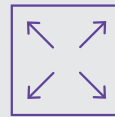

Expanded coverage area

Designed to extend coverage and manage the surgical site and surrounding soft tissue envelope

Pre-shaped designs for peel and place application



3M™ Prevena Restor™
Arthro•Form™ Dressing



3M™ Prevena Restor™
Axio•Form™ Dressing



3M™ Prevena Restor™
Bella•Form™ Dressing

Designed for flexibility and user customization



3M™ Prevena Restor™
Adapti•Form™ Dressing



The power behind 3M™ Prevena™ Therapy



3M™ Prevena™ Plus 125 Therapy Unit

One single-use negative pressure therapy unit compatible with all 3M™ Prevena™ Dressings.

Negative pressure features:

- Pre-set, continuous negative pressure therapy at -125 mmHg for up to 7 or 14 days (with dressing changes every 7 days)
- Disposable, single patient use
- Rechargeable battery

Specifications:

- Dimensions: Approx 8.9 × 16.3 × 5.49cm
- Weight with empty canister: 0.64lbs (0.29kg)

Prevena Dressings are also compatible with 3M traditional negative pressure therapy devices: 3M™ V.A.C.® Ultra Therapy Unit, 3M™ ActiV.A.C.® Therapy Unit



Proactive Risk Management with 3M™ Prevena™ Incision Therapy

Proactive Risk Management (PRM) with Prevena Therapy provides healthcare professionals with an evidence-based, standardized approach that helps to advance the standard of care for closed incision negative pressure therapy.



Prevena Central

Prevena Central is your one-stop platform for all things Prevena Therapy. Designed with busy healthcare professionals in mind, Prevena Central provides incision management resources that help advance the standard of care.

Visit [3M.com/PrevenaCentral](https://www.3m.com/PrevenaCentral) for more information.

For more information about implementing PRM in your practice, contact your 3M representative.

Additional customer resources:



Live clinical training and product support thousands of healthcare professionals trained annually



Product evaluation program



Clinical and reimbursement support hotlines



Centralized, on demand clinical and technical support

Ordering Information

SKU	Description	UOM
Therapy Devices		
PRE4000US	3M™ Prevena™ Plus 125 Therapy Unit – 7 day	Each
PRE4010	3M™ Prevena™ Plus 125 Therapy Unit – 14 day	Each
Dressings		
PRE1155US	3M™ Prevena™ Peel and Place Dressing – 13 cm	Case of 5
PRE1055US	3M™ Prevena™ Peel and Place Dressing – 20 cm	Case of 5
PRE3255US	3M™ Prevena™ Plus Peel and Place Dressing – 35 cm	Case of 5
PRE4055US	3M™ Prevena™ Plus Customizable Dressing	Case of 5
PRE5055	3M™ Prevena Restor™ Arthro●Form™ Dressing – 33 cm x 30 cm	Case of 5
PRE5155	3M™ Prevena Restor™ Arthro●Form™ Dressing – 46 cm x 30 cm	Case of 5
PRE5255	3M™ Prevena Restor™ Bella●Form™ Dressing – 21 cm x 19 cm	Case of 5
PRE5355	3M™ Prevena Restor™ Bella●Form™ Dressing – 24 cm x 24 cm	Case of 5
PRE5455	3M™ Prevena Restor™ Bella●Form™ Dressing – 29 cm x 27 cm	Case of 5
PRE5555	3M™ Prevena Restor™ Axio●Form™ Dressing – 29 cm x 28 cm	Case of 5
PRE6055	3M™ Prevena Restor™ Adapti●Form™ Dressing – 49 cm x 28 cm	Case of 5
Accessories		
PRE1095	3M™ Prevena™ 45 ml Canister	Case of 5
PRE4095	3M™ Prevena™ Plus 150 ml Canister	Case of 5
44001674	3M™ Prevena™ Plus 125 Therapy Unit Power Supply (available if needed)*	Each
413628	3M™ V.A.C.® Therapy System Power Cord (available if needed)*	Each
Kits		
PRE1001US	3M™ Prevena™ Incision Management System – 20 cm	Each
PRE1101US	3M™ Prevena™ Incision Management System – 13 cm	Each
PRE3201US	3M™ Prevena™ Plus Incision Management System – 35 cm	Each
PRE4001US	3M™ Prevena™ Plus Customizable Incision Management System	Each
PRE1121US	3M™ Prevena™ Duo Incision Management System – 13 cm/13 cm	Each
PRE3321US	3M™ Prevena™ Plus Duo Incision Management System – 13 cm/20 cm	Each
PRE5001	3M™ Prevena Restor™ Arthro●Form™ Incision Management System – 33 cm x 30 cm	Each
PRE5101	3M™ Prevena Restor™ Arthro●Form™ Incision Management System – 46 cm x 30 cm	Each
PRE5221	3M™ Prevena Restor™ Bella●Form™ Incision Management System – 21 cm x 19 cm	Each
PRE5321	3M™ Prevena Restor™ Bella●Form™ Incision Management System – 24 cm x 24 cm	Each
PRE5421	3M™ Prevena Restor™ Bella●Form™ Incision Management System – 29 cm x 27 cm	Each
PRE5501	3M™ Prevena Restor™ Axio●Form™ Incision Management System – 29 cm x 28 cm	Each
PRE6001	3M™ Prevena Restor™ Adapti●Form™ Incision Management System – 49 cm x 28 cm	Each

*Both 44001874 and 413628 are required to recharge the Prevena Plus Therapy Unit.

See why 3M™ Prevena™ Therapy is trusted by surgeons to help protect patients beyond the OR.

For more information or to request an evaluation, contact your 3M representative or visit 3M.com/PrevenaCentral.



Note: Specific indications, limitations, contraindications, warnings, precautions and safety information exist for these products and therapies. Please consult a clinician and product instructions for use prior to application. Rx only.

References:

1. Giles KA, Hamdan AD, Pomposelli FB, et al. Body mass index: surgical site infections and mortality after lower extremity bypass from the national surgical quality improvement program 2005-2007. *Ann Vasc Surg.* 2007;24:48-56.
2. Davenport DL, Zwischenberger BA, Xenos ES. Analysis of 30-day readmission after aortoiliac and infrainguinal revascularization using the American College of Surgeons national surgical quality improvement program data set. *J Vasc Surg.* 2011;60:1266-74.
3. Kuy S, Dua A, Desai S, Dua A, Patel B, Tondravi N, et al. Surgical site infections after lower extremity revascularization procedures involving groin incisions. *Ann Vasc Surg.* 2014;28:53-8.
4. Ozaki CK, Hamdan AD, Barshes NR, et al. Prospective, randomized, multi-institutional clinical trial of a silver alginate dressing to reduce lower extremity vascular surgery wound complications. *J Vasc Surg.* 2015;61:419-27.e1.
5. Wiseman JT, Fernandes-Taylor S, Barnes ML, et al. Predictors of surgical site infection after hospital discharge in patients undergoing major vascular surgery. *J Vasc Surg.* 2012;62:1023-31.e5.
6. Nguyen L, Leya GA, Hevelone ND, et al. Prospective cost analysis and implications of wound complications in lower extremity vascular surgery procedures. *J Vasc Surg.* 2014;60(3):813.
7. Zimlichman E, Henderson D, Tamir, et al. Health care-associated infections a meta-analysis of costs and financial impact on the U.S. health care system. *JAMA Intern Med.* 2013;173(22):20-46.
8. Urban JA. Cost analysis of surgical site infections. *Surg Infect.* 2006.1:7(Suppl 1):S19-S22.
9. Magill SS, Edwards JR, Bamberg W, et al. Multistate point-prevalence survey of health care-associated infections. *N Engl J Med.* 2014;370(13):1198-1208.
10. Zhan C, Miller MR. Excess length of stay, charges, and mortality attributable to medical injuries during hospitalization. *JAMA.* 2003;290(14):1868-1874.
11. Wilkes RP, Kilpadi DV, Zhao Y, et al. Closed Incision Management With Negative Pressure Wound Therapy (CIM): Biomechanics. *Surgical Innovation.* 2012;19(1):67-75.
12. Haridas B, Kieswetter K, Haggerty M. Test Report: Negative Pressure Therapy on Intact Skin: Poroelastic Finite Element Modeling of Interstitial Fluid Pressures. DOC-0000049240 Rev A, 1-1.3. 6-27-2019. San Antonio, TX, KCI. Ref Type: Report.
13. Willy C, Agarwal A, Andersen CA, et al. Closed incision negative pressure therapy: International multidisciplinary consensus recommendations. *Int Wound J.* 2017;14(2):385-398.
14. Antoniou G, Onwuka C, Antoniou S et al. Meta-analysis and trial sequential analysis of prophylactic negative pressure therapy for groin wounds in vascular surgery. *J Vasc Surg* 2019; 70 (5):1700-1710.



3M Company
2510 Conway Ave
St. Paul, MN 55144 USA

Phone 1-800-275-4524 (NPWT products)
1-800-228-3957
Web 3M.com/medical

© 2023 3M. All rights reserved. 3M and the other marks shown are marks and/or registered marks. Unauthorized use prohibited. 70-2013-1639-8