

Put the power of two to work for you.

The strengths of 3M™ Kerramax Care™ Super-Absorbent Dressing and 3M™ Coban™ 2 Two-Layer Compression System can help you manage venous leg ulcers (VLU).



A combination your patients can count on.

The combination of 3M™ Kerramax Care™ Dressing and 3M™ Coban™ 2 Compression System is designed to help clinicians and patients with the management of VLU. The low profile of Kerramax Care Dressing and its ability to absorb and retain exudate even under compression, makes it a great partner with compression therapy.^{1,2} The Coban 2 Layer Compression System is a two-layer compression bandage system that is designed to deliver therapeutic compression required for treating patients with VLUs.³

Wounds International recently published eight case studies evaluating the clinical performance of Kerramax Care Dressing and Coban 2 Two-Layer Compression System. Outcomes show that patients with VLUs being treated in different care settings were satisfied with the combined treatment. It was reported that Kerramax Care Dressing stayed in place during wear time under compression, retained exudate and no strikethrough or leakage was observed.¹

1 3M™ Kerramax Care™ Super-Absorbent Dressing

Chronic, highly exuding wounds are difficult for both clinician and patient. Excess fluid can make achieving an optimum moisture balance difficult and can lead to maceration.

Kerramax Care Dressing with advanced Exu-Safe Technology absorbs and retains high levels of serous and viscous fluid away from the periwound skin and wound bed, helping to reduce skin maceration, even under compression.^{1,4}



3M™ Kerramax Care™ Super-Absorbent Dressing

The high absorption and retention of Kerramax Care Dressing can enable fewer dressing changes and minimize disturbance to the wound bed.^{4,5}

A positive patient experience can lead to reduced stress and anxiety when dealing with chronic wounds, which can help to reduce pain and improve patient compliance with treatment. Kerramax Care Dressing is designed for patient comfort.^{4,5}

Patients

70%

Scored the dressing between 8-10 highlighting a **high level of comfort and positive patient experience.**⁴

n=101



Happy to continue

90%

90% of responses indicated would be happy to continue using Kerramax Care Dressings as first choice for the management of highly exuding wounds.⁴



Key benefits




Absorbs and retains high levels of exudate^{4,5}



Soft, conformable and foldable²



Wicks vertically, then spreads out horizontally to minimize bulkiness

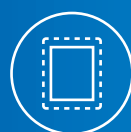

Low-profile, heat-sealed border



Easy to apply (using either side)



Can be used under compression⁵



Can be left in place for seven days



Venous leg ulcers affect 1% to 2% of the population, with the prevalence increasing $\leq 4\%$ for those aged >65 years.⁷

A monthly surface area reduction of 30%  provides a baseline healing rate for VLUs managed with compression therapy and advanced dressings.⁸



2 3M™ Coban™ 2 Two-Layer Compression System and 3M™ Coban™ 2 Lite Two-Layer Compression System

The Coban 2 Compression System and Coban 2 Lite Compression System provide effective compression therapy, which has been shown to contribute to effective VLU management through edema reduction, decreased pain and improvement in a patient's daily activities.^{9,10,11}

Coban 2 compression system is easy to apply and remove, is designed to stay in place,^{9,10} and allows for normal footwear, increasing patient compliance and the potential for more effective treatment.^{9,11}



3M™ Coban™ 2
Two-Layer Compression System



3M™ Coban™ 2 Lite
Two-Layer Compression System

49%

of VLUs can be **healed by compression**¹²



Coban 2 Compression System is also effective at reducing lymphedema and chronic edema

Key benefits



Fast and easy applications



Therapeutic compression that stays in place



Comfortable to improve patient compliance



Not made with natural rubber latex



Conforms to patient anatomy



Provides consistent compression



Helps enable a more normal lifestyle

Product ordering information

3M™ Kerramax Care™ Super-Absorbent Dressing

3M Code	Size	Each Carton/Box	HCPCS Code
PRD500-025	5 cm x 5 cm (2 in x 2 in)	10 ea/bx	A6196
PRD500-050	10 cm x 10 cm (4 in x 4 in)	10 ea/bx	A6196
PRD500-100	13.5 cm x 15.5 cm (5 in x 6 in)	10 ea/bx	A6197
PRD500-120	10 cm x 22 cm (4 in x 9 in)	10 ea/bx	A6197
PRD500-240	20 cm x 22 cm (8 in x 9 in)	10 ea/bx	A6197
PRD500-380-B10	20 cm x 30 cm (8 in x 12 in)	10 ea/bx	A6198
PRD500-600-B10	20 cm x 50 cm (8 in x 20 in)	10 ea/bx	A6198

View case studies
to see how these
products provide
optimal environments for
managing VLU:



3M™ Coban™ 2 Two-Layer Compression System and 3M™ Coban™ Lite Two-Layer Compression System

Coban 2 Layer Compression System, with the purple core and packaging, provides high compression and is ideal for treating the majority of patients with ABPI greater than or equal to 0.8. Coban 2 Lite Compression System, with the green core and packaging, provides a reduced resting pressure compared to Coban 2 Layer Compression System. Designed for those limbs with small circumferences (e.g. fingers, arms), for patients less tolerant of compression therapy, including those who have mixed etiology, are new to compression or where tolerance is not known. Suitable for ABPI 0.5 and above.

Description	3M Code	Roll Dimensions	Rolls/Box	Boxes/Case	HCPCS Code
3M™ Coban™ 2 Two-Layer Compression System for patients with ABPI ≥ 0.8	2094N	Roll 1 – 4 in x 2.9 yd (10 cm x 2.7 m) Roll 2 – 4 in x 3.8 yd (10 cm x 3.5 m)	2 rolls, 1 of each layer	8	A6441 A6452
3M™ Coban™ 2 Two-Layer Compression System for patients with ABPI ≥ 0.8	2094XL	Roll 1 – 4 in x 3.8 yd (10 cm x 3.5 m) Roll 2 – 4 in x 4.9 yd (10 cm x 4.5 m)	2 rolls, 1 of each layer	8	A6441 A6452
3M™ Coban™ 2 Two-Layer Compression System for patients with ABPI ≥ 0.8	20096	Roll 1 – 6 in x 3.8 yd (15 cm x 3.5 m) Roll 2 – 6 in x 4.9 yd (15 cm x 4.5 m)	2 rolls, 1 of each layer	8	
3M™ Coban™ 2 Lite Two-Layer Compression System for patients with ABPI ≥ 0.5	2794N	Roll 1 – 4 in x 2.9 yd (10 cm x 2.7 m) Roll 2 – 4 in x 3.4 yd (10 cm x 3.2 m)	2 rolls, 1 of each layer	8	A6441 A6452

NOTE: Specific indications, contraindications, warnings, precautions and safety information exist for these products and therapies. Please consult a clinician and product Instructions for Use prior to application. This material is intended for health care professionals. Rx only.

References:

- Hughes, M A, et al. 3M™ Kerramax Care™ Super Absorbent Dressing and 3M™ Coban™ 2 Two-Layer Compression System case series supplement. Wounds International (2022), London, UK.
- Jackson S, Lovett J, Stephenson C. An assessment of the conformability of superabsorbent dressings. Poster presented at: Wounds UK Annual Conference 2016, UK.
- Vowden P, Kerr A, Mosti G: Demystifying mild, moderate and high compression systems -when and how to introduce "lighter" compression. Position document Wounds International. Nov 2020.
- Hughes, M. A large-scale evaluation of managing moderate and highly exuding wounds in the community. *Wounds UK*. 2017;13(3):78-85.
- Jones J, Barraud J. An evaluation of Kerramax Care in the management of moderate to heavily exuding wounds. *Br J Community Nurs*. 2014 Mar; Suppl: S8, S50-53
- Singh G, Byrne C, Thomason H, McBain A. Investigating the microbial and metalloprotease sequestration properties of superabsorbent wound dressings. *Sci Rep*. 2022; 12 (1): 4747
- Schul MW, Melin MM, Keaton TJ. Venous leg ulcers and prevalence of surgically correctable reflux disease in a national registry. *J Vasc Surg Venous Lymphat Disord*. 2023 May;11(3):511-516. doi: 10.1016/j.jvs.2022.11.005. *Epub* 2023 Jan 19. PMID: 36681297.
- Rajhathy, Erin M.; Murray, Heather D.; Roberge, Veronica A.; Woo, Kevin Y. Healing Rates of Venous Leg Ulcers Managed With Compression Therapy: A Secondary Analysis of Data. *Journal of Wound, Ostomy and Continence Nursing* 47(5):p 477-483, September/October 2020. | DOI: 10.1097/WON.0000000000000693
- Mosti G, Crespi A, Mattaliano V: Comparison Between a New, Two-component Compression System With Zinc Paste Bandages for Leg Ulcer Healing: A Prospective, Multicenter, Randomized, Controlled Trial Monitoring Sub-bandage Pressures *WOUNDS* 2011;23(5):126-134
- Moffatt C. et al: A preliminary randomized controlled study to determine the application frequency of a new lymphoedema bandaging system, *British Journal of Dermatology*, Volume 166, Issue 3, March 2012, Pages 624-632
- Moffatt C et al.: A randomised controlled 8-week crossover clinical evaluation of the 3M™ Coban™ 2 Layer Compression System versus Profore™ to evaluate the product performance in patients with venous leg ulcers
- Guest JF, Fuller GW. Relative cost-effectiveness of three compression bandages in treating newly diagnosed venous leg ulcers in the UK. *J Wound Care*. 2023 Mar 2;32(3):146-158.



Solventum Medical Surgical
3M Center, Building 275,
2519 Conway Avenue East
St. Paul, MN 55144-1000

Phone 1-800-275-4524 (NPWT products)
1-800-228-3975
Web Solventum.com



Contact your local Solventum representative
for education and more information.

© Solventum 2024. All Rights Reserved. Solventum and the S logo are trademarks of Solventum or its affiliates. 3M and the 3M logo are trademarks of 3M. Other trademarks are the property of their respective owners. US_70-2013-1661-0