



# Curoso™

Disinfecting Port  
Protectors

How to use


## 3M™ Curoso™

Disinfecting Cap for Needleless Connectors

**Disinfects in one minute – protects needleless connectors for up to seven days.**



**Peel**  
the foil from the 3M™ Curoso™ Disinfecting Cap.

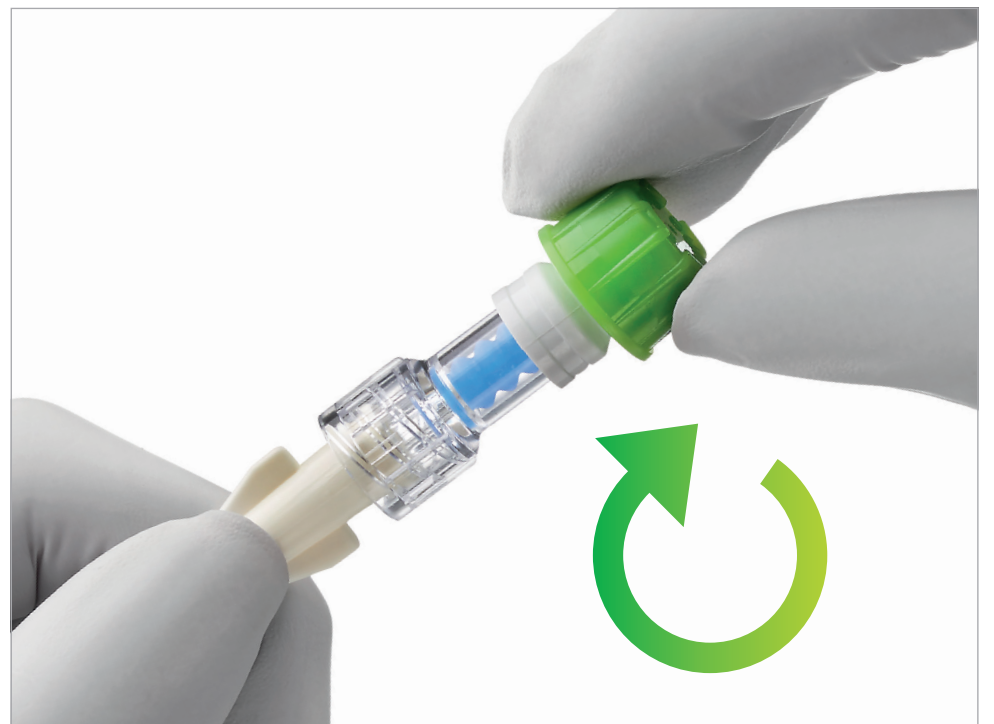

**Push and twist**  
the Curoso disinfecting cap clockwise onto the end of the needleless connector.

- ▶ If needleless connector is heavily soiled, clean with 70% isopropyl alcohol swab before using cap
- ▶ Do not use cap if seal is broken, torn, punctured, or sponge appears dry
- ▶ Cap must remain on needleless connector for a minimum of one minute and may remain on for seven days if not removed
- ▶ The needleless connector can be accessed immediately after removing the Curoso disinfecting cap that has been in place for a minimum of one minute
- ▶ Always place a new Curoso disinfecting cap on needleless connector after each use
- ▶ Dispose of the Curoso disinfecting cap after every use
- ▶ Follow hospital protocol for disinfecting needleless connectors, if performing sequential accesses

**Warning:** to avoid potential for injury, use only on needleless connectors.

**Caution:** potential choking hazard.