


3M™ Curoos™ Stopper

Disinfecting Cap for Open Female Luers

Disinfects in one minute – protects open female luers for up to seven days.

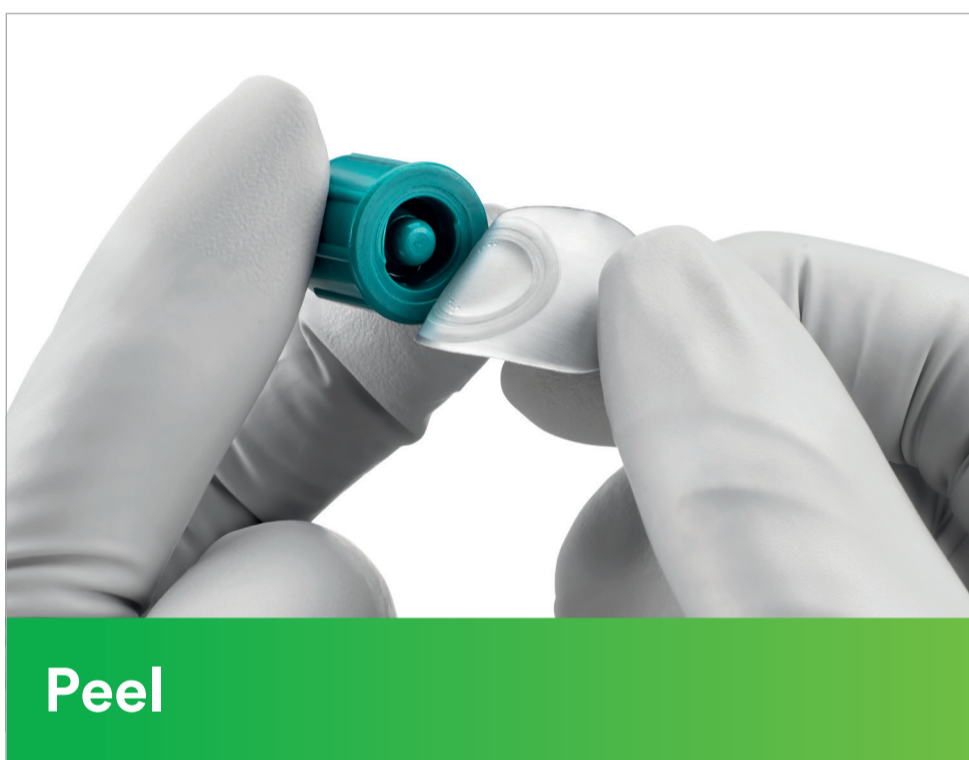

- ▶ Verify stopcock valve or catheter clamps are in the closed position prior to using the 3M™ Curoos™ Disinfecting Cap for Open Female Luers
- ▶ If open female luer is visibly soiled, clean with 70% Isopropyl Alcohol swab before using cap
- ▶ Do not use cap if the seal is broken, torn, or punctured
- ▶ Detach the cap from the foil
- ▶ Once foil is removed, push and twist device clockwise onto the end of the open female luer until tightly connected
- ▶ Cap must remain in place for a minimum of one minute and may remain on for seven days if not removed
- ▶ The open female luer can be accessed immediately after removing the Curoos Stopper disinfecting cap that has been in place for a minimum of one minute
- ▶ Discard cap after each use; do not reuse cap
- ▶ New cap should be placed on open female luer after each use
- ▶ Follow hospital protocol for disinfecting open female luers, if performing sequential accesses

Warning: to avoid potential for injury, use only open female luers and ensure the cap is securely tightened.

Caution: potential choking hazard.

