



Cuрос™

Disinfecting Port
Protectors

How to use

3M™ Cuрос™ for Tego®

Disinfecting Cap for Tego® Hemodialysis Connectors

Disinfects in one minute – protects for up to seven days.



Peel

the foil from the 3M™ Cuрос™ Disinfecting Cap for Tego® Hemodialysis Connector.



Push and twist

the Cuрос disinfecting cap for Tego® hemodialysis connector clockwise onto the end of the hemodialysis connector.

- ▶ If needleless connector is heavily soiled, clean with 70% isopropyl alcohol swab before using cap
- ▶ Do not use cap if seal is broken, torn, punctured, or sponge appears dry
- ▶ Cap must remain on needleless connector for a minimum of one minute and may remain on for seven days if not removed
- ▶ The needleless connector can be accessed immediately after removing the Cuрос disinfecting cap for Tego® hemodialysis connector that has been in place for a minimum of one minute
- ▶ Always place a new Cuрос disinfecting cap for Tego® hemodialysis connector on hemodialysis connector after each use
- ▶ Dispose of the Cuрос disinfecting cap for Tego® hemodialysis connector after every use
- ▶ Follow hospital protocol for disinfecting needleless connectors, if performing sequential accesses

Warning: to avoid potential for injury, use only on Tego® Hemodialysis Connectors.

Caution: potential choking hazard.